

TIM TRIO Table Controls

Landmarking and Home positioning





TRIO

TIM TRIO




TRIO and TIM TRIO Similarities

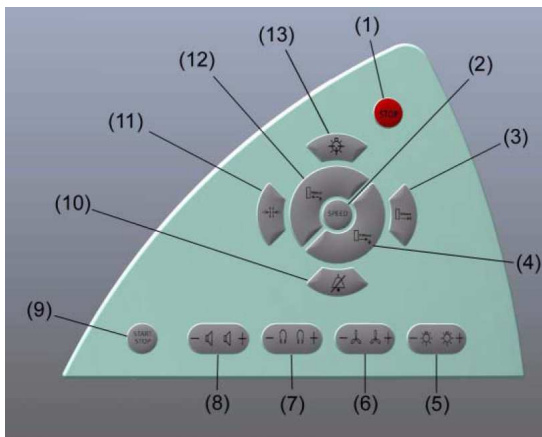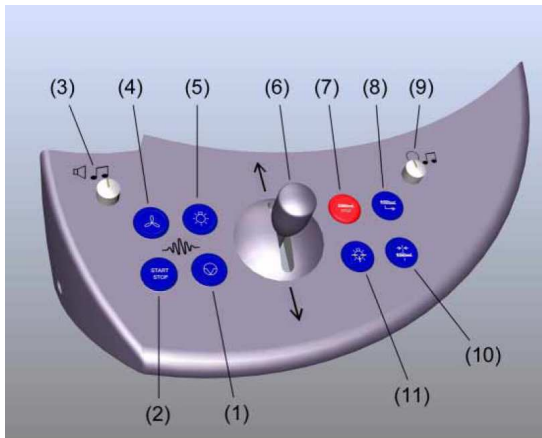

The table control icons on the TIM TRIO are exactly the same as the table control ions on the TRIO system. The TIM TRIO does not use a joystick to move the table up/down and in/out. You will use buttons instead to move the table up/down and in/out , which is very simple. With the TIM TRIO, always register your subject first before positioning and landmarking your subject. If you position and landmark first and then register, the table location will change without notice.

Trio Table Control

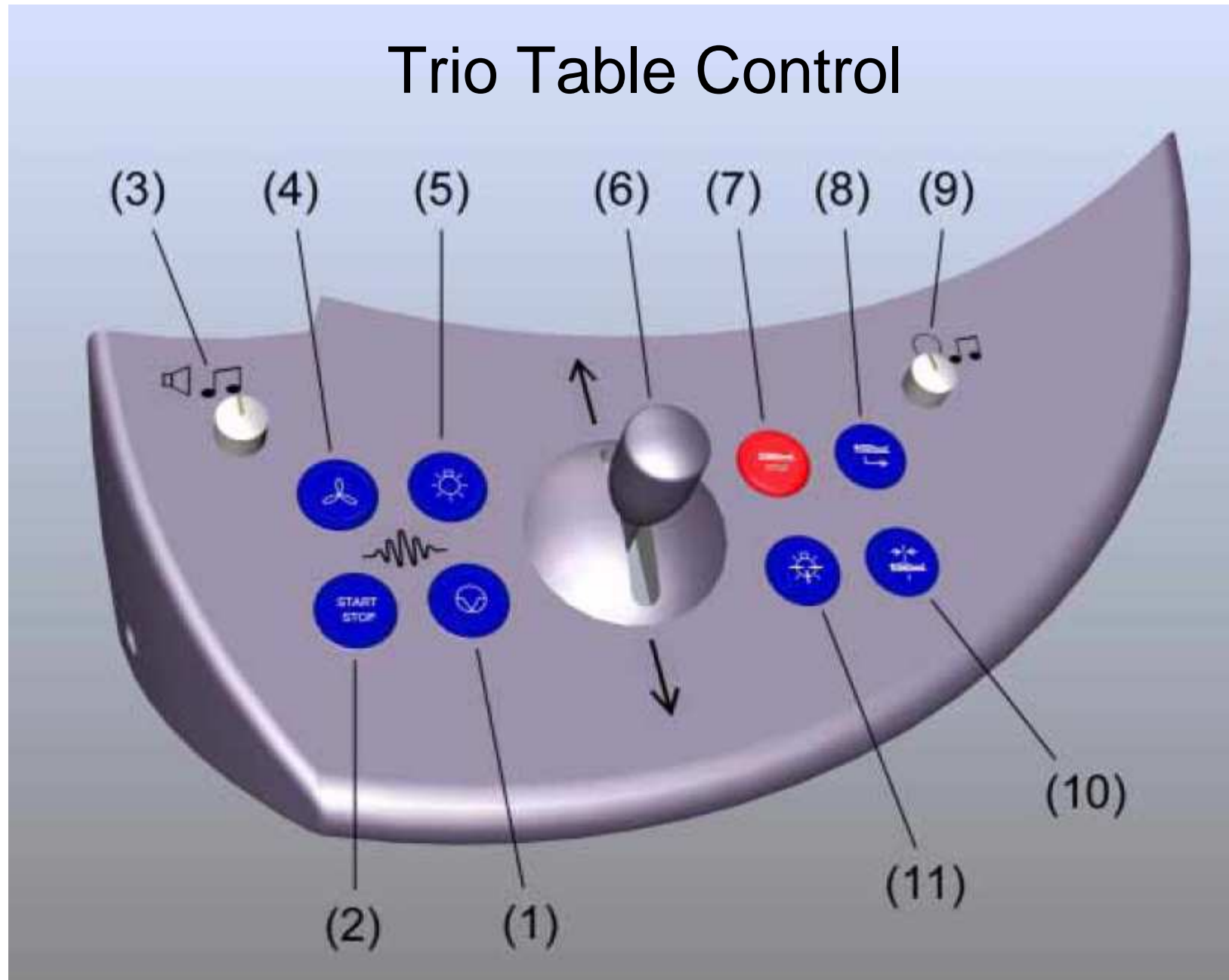


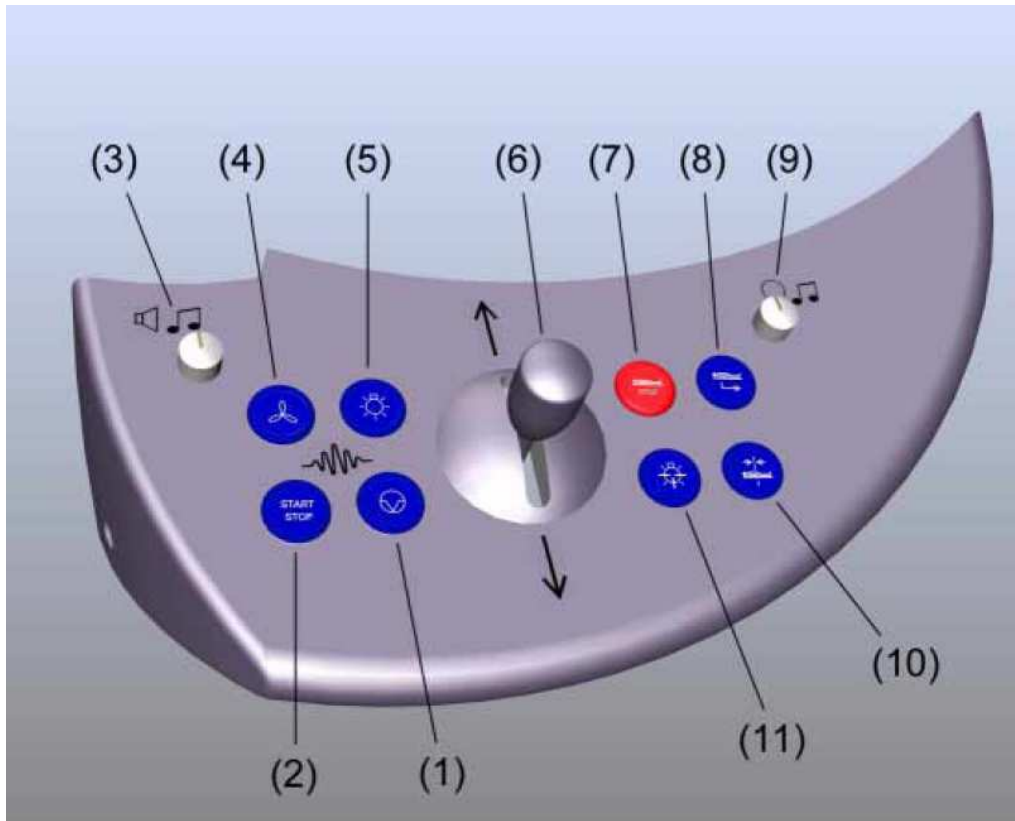


Table Control on Trio

1. Future Use
2. Start / Stop Scan (not used)
3. Volume for external speaker
4. Bore fan
5. Bore light
6. Joy stick – table up/down & in/out
7. Emergency table brake release
8. Table home
9. Volume for head phones
10. Table to center scan position
11. Landmark laser light localizer

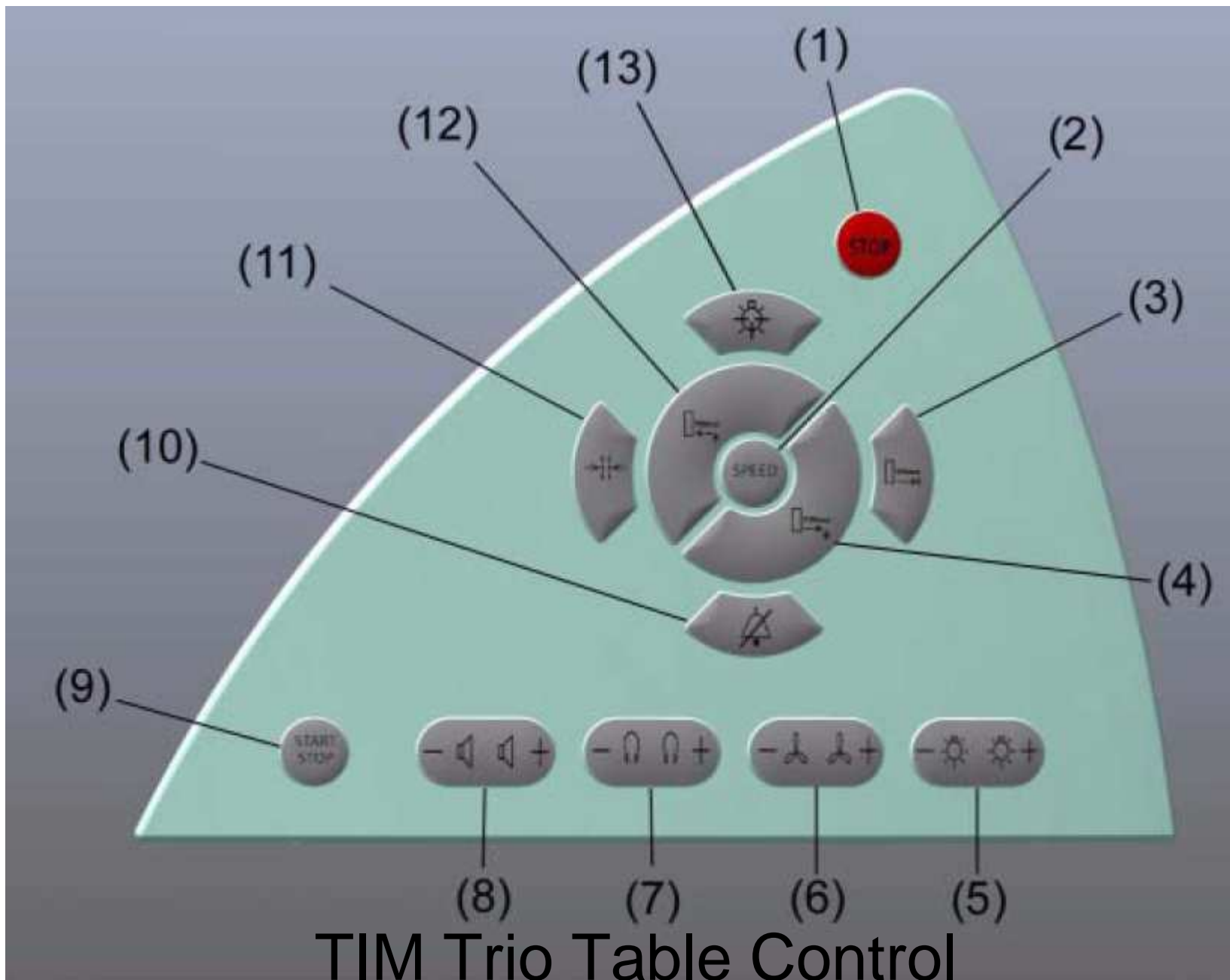

TIM Trio Table Control


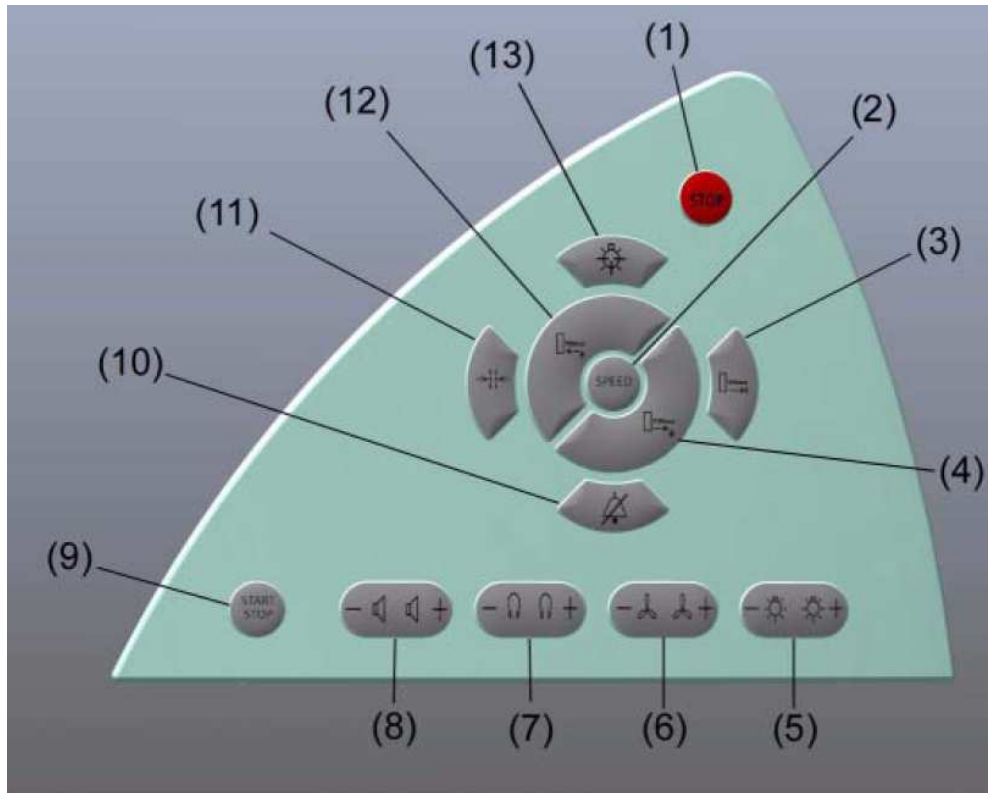


Table controls on TIM Trio

1. Emergency table brake release
2. Table speed control
3. Table home
4. Table out & down
5. Bore light
6. Bore fan
7. Volume headphones
8. Volume speaker
9. Start / Stop Scan (not used)
10. Future use
11. Table to center scan position
12. Table up & in
13. Landmark laser light localizer

Procedure for table landmarking

1. **Always register your subject first before landmarking. If you landmark and then register the scan location will move from isocenter without notice.**
2. Press button #12 to raise table to upper most position
3. Press button #12 to move anatomy under laser light. Note: Pressing button #2 with button #12 increases table travel speed.
4. Press button #13 to activate laser light. Center to anatomy. Press button # 13 to deactivate laser light.
5. Press either button #11 to move to isocenter or buttons #12 & #2 until table location = 0 on LCD screen.
6. You are now ready to scan.

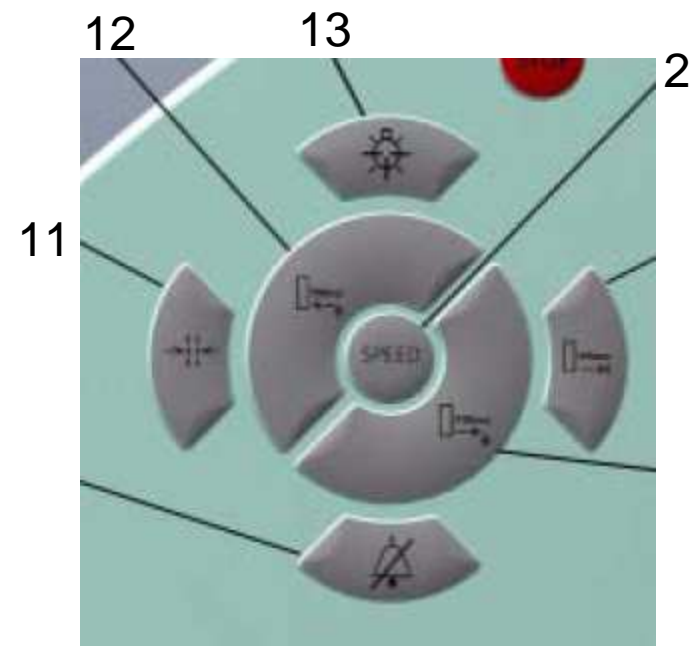

Procedure for returning table to Home position

1. Press either button #3 to move table to Home position or buttons #4 & #2 until table is all the way out.
2. Press button #4 to move the table down to a level where your subject can comfortably get down off the table.

