
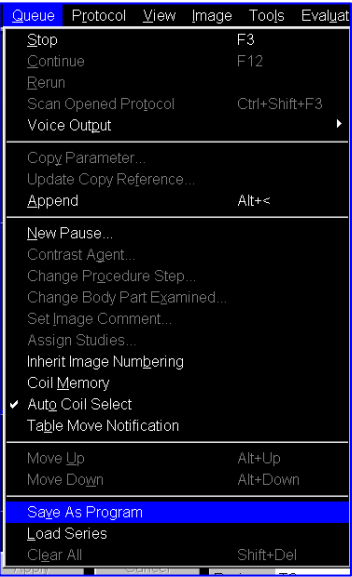
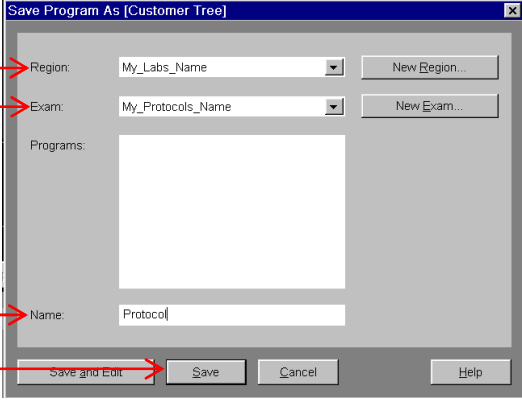
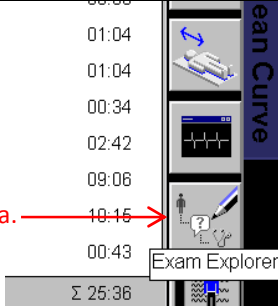
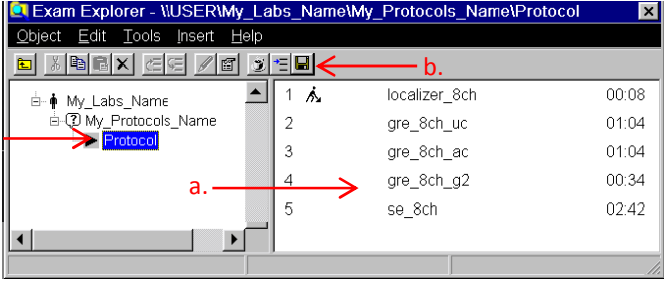


Saving, Editing, & Exporting Your Protocol to USB Storage

<p>Step 1</p> <ol style="list-style-type: none"> Click on "Queue" on the menu toolbar Click on "Save As Program". 	 <p>a. → Queue Protocol View Image Tools Evaluate</p> <p>b. → Save As Program</p>															
<p>Step 2</p> <ol style="list-style-type: none"> Go to "Region" field and click down arrow. Select your lab. Go to "Exam" field. Click down arrow if the study is being modified and select that study. If this is a new study, click on "New Exam". A pop-up menu will appear. Enter the name of the study and click "Ok". Go to "Name" field. Enter the name of your protocol. Click on "Save". 	 <p>a. → Region: My_Labs_Name</p> <p>b. → Exam: My_Protocols_Name</p> <p>c. → Name: Protocol</p> <p>Save and Edit Save Cancel Help</p>															
<p>Step 3</p> <ol style="list-style-type: none"> Click on "Exam Explorer" pictogram to edit protocol. Find your lab and click on your protocol to open it. 	 <p>a. → Exam Explorer</p>															
<p>Step 4</p> <ol style="list-style-type: none"> To edit your protocol drag and drop a sequence(s) to change the location position. Delete sequence(s) that are not needed. To add comments or change the name of the sequence, right click on the sequence, click on "Properties" and add or edit text. Save protocol by clicking on floppy disk icon. Right click on your protocol. A menu will pop-up. 	 <p>b. → [Floppy Disk Icon]</p> <p>c. → Protocol</p> <p>a. → [Arrow pointing to sequence 4]</p> <table border="1" data-bbox="1063 1669 1453 1816"> <tr><td>1</td><td>localizer_8ch</td><td>00:08</td></tr> <tr><td>2</td><td>gre_8ch_uc</td><td>01:04</td></tr> <tr><td>3</td><td>gre_8ch_ac</td><td>01:04</td></tr> <tr><td>4</td><td>gre_8ch_g2</td><td>00:34</td></tr> <tr><td>5</td><td>se_8ch</td><td>02:42</td></tr> </table>	1	localizer_8ch	00:08	2	gre_8ch_uc	01:04	3	gre_8ch_ac	01:04	4	gre_8ch_g2	00:34	5	se_8ch	02:42
1	localizer_8ch	00:08														
2	gre_8ch_uc	01:04														
3	gre_8ch_ac	01:04														
4	gre_8ch_g2	00:34														
5	se_8ch	02:42														

Step 5

- a. Click on "Print" and another menu will pop-up.

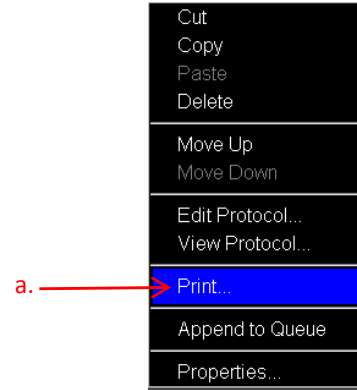

Step 6

- a. Insert your USB storage into the Siemens computer USB port.
- b. In the "File name" field enter F:\your_file_name.pdf
- c. Click on "Protocols and table of contents".
- d. Click "OK". Your protocol will be saved as a PDF file. Eject your USB storage.

