

The ShotBlocker® uses blunt contact points to distract patients 2 years old and up from the potential pain of needle procedures.

Overview:

- Select injection site and prep skin as per protocol.
- Hold the ShotBlocker® so that the blunt contact points touch the patient’s skin at the injection site.
- Press ShotBlocker® firmly against the skin. Do not move the ShotBlocker® until the injection has been completed.
- Administer injection through or near the ShotBlocker® opening. If more than 20 seconds lapse between the placement of ShotBlocker® and the procedure, remove the ShotBlocker® from the skin and repeat the process.
- The ShotBlocker® is for single-patient use. Patients can keep for future use. Wash with soap and water between uses.

When introducing to patients:

- Explain how the blunt contact points work to block the pain from the injection.
- Let the patient feel the tool and show how it will be used.

.....

Medline ordering number: HCI8050

Product information: Bionix.com/shotblocker.html

Website: <https://ucdavis.health.comfort>

