

Steps	HCP
<p><b>1. Dressed and ready to perform patient care</b></p> <ul style="list-style-type: none"> <li>○ Hair tied back</li> <li>○ Jewelry, , etc. removed and pockets empty (may keep Vocera)</li> <li>○ Shoes should have a back strap or heel.</li> <li>○ Wear mask under PAPR hood if universal masking is in effect per current policy.</li> </ul>	<input type="checkbox"/>
<p><b>2. Ensure all PPE available and check for integrity and correct sizing</b></p> <ul style="list-style-type: none"> <li>○ Gloves</li> <li>○ Isolation gown</li> <li>○ PAPR unit (turbine, belt, headcover, breathing tube)</li> </ul> <p><b>IMPORTANT:</b> ensure filter cover on turbine is snapped into place and secure</p>	<input type="checkbox"/>
<b>3. Perform hand hygiene</b>	<input type="checkbox"/>
<p><b>4. Prepare PAPR unit -refer to PAPR reference on PAPR cart for details.</b></p> <ul style="list-style-type: none"> <li>● Ensure battery is fully charged</li> <li>● Connect battery to turbine</li> <li>● Check the flow of the turbine level G</li> <li>● Connect breathing tube to turbine</li> <li>● Check flow of breathing tube connected to turbine level G</li> <li>● Place turbine around waist – adjust to fit</li> <li>● Drape/dangle tubing over shoulder</li> </ul>	<input type="checkbox"/>
<b>5. Don gown if needed for patient specific isolation precautions. Ensure gown does not impede air intake of the turbine.</b>	<input type="checkbox"/>
<b>6. Connect tubing to headcover or helmet and don headcover</b>	<input type="checkbox"/>
<b>8. Don gloves if needed for patient specific isolation precautions</b>	<input type="checkbox"/>

If there are issues with the PAPR functionality, sequester the unit including turbine and battery and call central processing x42550 or "CPU charge" on Vocera.